



IntelConnect EV

IntelViewer access via Web Browser

Web Browser URL:
<https://pacs.niddriexray.com.au>

Intelconnect EV Quick
Reference Shortcut Sheet

NXR IntelConnect EV (Enhanced Viewer) provides physicians with zero-footprint access to Niddrie Xray IntelViewer PACS. It allows referrers quick real-time access to patient images and reports on tablets, smartphones and web browsers.

Unlike IntelViewer, whereby an application is required to be installed onto the workstation, IntelConnect is accessible using a web browser via desktop, mobile or the iOS App.

IntelConnect EV grants access anywhere, anytime.

To view online:

Open any web browser

Enter; <https://pacs.niddriexray.com.au>

Use your IntelViewer username and password to log in

Search patients using the top search bar by; Surname, First Name OR Patient ID

Select the desired patient to view all images and reports

The patient report will display to the left, images to the right

Click on the image you wish to view, it will open in the IntelConnect Interface

To download the App:

Go to the App Store on your smart phone or tablet

Search 'IntelConnect'

Download the free App

Open the App

Select New Account

Enter server: <https://pacs.niddriexray.com.au>

Enter your IntelViewer username and password

Search the desired patient

Use the scroll bar at the bottom of the screen to scroll through the images